Digital Thermometer

Model UT-103

Instruction Manual Original

English

Mode d'Emploi

Français

Manual de Instrucciones

Español

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Itaiano

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1WMPD4002219A

A&D Company, Limited

3-23-14 Higashi-Ikebukuro, Toshima-ku, Tokyo, JAPAN
Telephone: [81] (3) 5391-6132 Fax: [81] (3) 5391-6148

Vega Technologies Inc.

11F-13, 100 Chang-chun Road, Taipei, Taiwan
Telephone: [886] (2) 2541-6996 Fax: [886] (2) 2521-3803

Biolife, Ltd

203, Mesogion, AV.P.C. 115 25 Athens, Greece
Telephone: [30] (210) 6718743 Fax: [30] (210) 6718745

Preliminary remarks

- This device conforms to the European Directive 93/42 EEC for Medical Products. This is proved by the CE conformity marking **CE**⁰¹⁹⁷ (0197: The identification number of the Notified Body)
- Environment for use: The device is designed for use indoors.

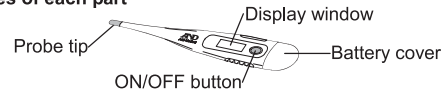
Precautions

- Precision components are used in this device. Extreme temperature, humidity, direct sunlight, shock or dust should be avoided.
- Measurements may be affected if the device is used near a television, microwave oven, cellular telephone, X-ray machine or other devices that emit electromagnetic waves.
- The thermometer should only be used under the supervision of an adult.
- Used equipment, parts and batteries are not treated as ordinary household waste, and must be disposed of according to the guidelines in your area.
- Before and after each use, clean the thermometer gently using a soft cloth moistened with water-diluted isopropyl alcohol or cold soapy water.
- Do not immerse the thermometer in water.
- Do not sterilize the thermometer by boiling or gas or steam autoclaves.

Symbols

Symbols	Function / Meaning	Symbols	Function / Meaning
	Standby and power on.		Direct current
SN	Serial number		Date of manufacture
	Type B: The device is designed to provide special protection against electrical shocks.		
	Last reading stored in memory		
	The battery is low. Replace the battery with a new one.		
H	The reading is over 43 °C.		Do not use the thermometer to measure the temperature of the object.
L	The reading is below 32 °C.		
CE ⁰¹⁹⁷	EC directive medical device label		WEEE label
	Manufacturer		EU-representative
	Consult instructions for use		

Names of each part

How to take your temperature

Note: Do not walk, run or talk during temperature measurement.

- Press the ON/OFF button to turn the thermometer on. The thermometer beeps and displays **100.0°C** (or **100.0°F**) for about 2 seconds. The last temperature taken is displayed with an "M" mark in the right corner for about 1 second.
- On the display, 37.0°C (or 98.6°F) appears.

Switching between Fahrenheit (°F) and Celsius (°C):

- Press the ON/OFF button to turn the thermometer off.
- Press and hold the ON/OFF button for 5 seconds. The display changes to the other measurement scale.
- When "Lo" and flashing °C (°F) are displayed, the thermometer is ready for temperature measurement. ("Hi" or room temperature is displayed when the room temperature reaches a certain temperature as shown below.)

Room temperature	<32.0°C (89.6°F)	32.0°C to 42.9°C	>42.9°C (109.2°F)
Display	Lo	Room temperature	Hi

Note: The thermometer can not measure a person's temperature when the room temperature is higher than the body temperature.

- The thermometer beeps when the measurement is completed.
- Press the ON/OFF button to turn the thermometer off. (The thermometer automatically turns off, about 8 minutes ± 2 minutes after use.)

English 1

Taking an oral temperature

Place the probe tip under the tongue as near as possible to a heat pocket.

Approximate measuring time: 1 minute.

Heat pocket

Taking an axillary temperature (under the arm)

- Wipe the underarm with a dry towel.
- Place the probe tip under the arm so the tip touches the skin with the thermometer perpendicular to the body.
- Position the arm across the chest so the probe tip is well covered by the arm.

Approximate measuring time: 2-3 minutes.

Note: Axillary temperatures are generally 0.5°C /1.0°F lower than oral readings.

Taking a rectal temperature

- Apply a water-soluble lubricant to the probe tip.
- Gently insert the probe (maximum 1/2") into the rectum.

Approximate measuring time: 1 minute.

Note: Rectal temperatures are generally 0.5°C /1.0°F higher than oral readings.

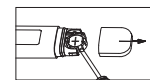
Maintenance

- Store the thermometer in the protective case when not in use.
- Do not store the thermometer where it will be exposed to direct sunlight, dust or humidity. Avoid extreme temperatures.
- Do not drop or subject the thermometer to strong shocks.
- Do not attempt to disassemble the thermometer, except to replace the battery.

Replacing the battery

Replace the battery with a new one when appears in the lower right corner of the display. Battery type: 1.55V, LR41 alkaline or SR41 silver oxide type

- Remove the battery cover.
- Use a non-metal pointed tool to remove the battery and insert the new battery with the positive (+) side facing up.
- Slide the battery cover back into place.

Note: Keep the battery out of the reach of children.

Be careful not to swallow the battery cover accidentally.

Specifications

Measurement range	32.0°C (89.6°F) to 42.9°C (109.2°F)
Low temperature display	Temperature<32.0°C (89.6°F): display: Lo°C (°F)
High temperature display	Temperature>42.9°C (109.2°F): display: Hi°C (°F)
Temperature display	0.1°C per (0.1°F per)
Accuracy	35.5°C (95.9°F) to 42.0°C (107.6°F): ±0.1°C (0.2°F) Other ranges: ±0.2°C (0.3°F)
Fever beep Sound	>37.5°C (99.5°F) 30 short beep sounds
Normal beep sound	≤37.5°C (99.5°F) 10 long beep sounds
Auto power off	About 8 minutes ± 2 minutes
Battery	LR41 or SR41
Battery life	2 years
Dimensions	130.36 x 18.67 x 10.55 mm
Operating conditions	Temperature: 10°C to 40°C (50.0°F to 104.0°F) Humidity: 30%RH to 85%RH
Storage conditions	Temperature: -10°C to 60°C (14.0°F to 140.0°F) Humidity: 25%RH to 90%RH

EMC table information is listed on our website:
http://www.aandd.jp/products/manual/medical/emc_en.pdf

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