


# **A&D** Instruction Manual Addendum

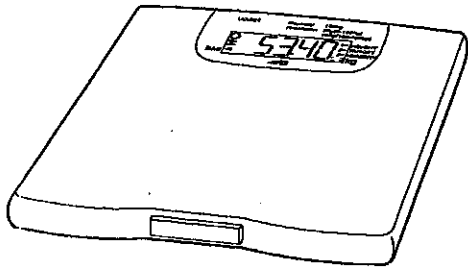

## Precision Personal Health Scale Model UC-321P

### Tracking Your Weight with Your Personal Computer

- UC-321P can monitor your weight by recording the data to a personal computer and displaying the data on the computer display via the "Weight Diary" Windows™ software.
- You can track your weight data and BMI values on the personal computer display by graph or tabular format.

#### Accessories

- UC-321 Instruction manual and addendum
- Connecting cable
- CD-ROM "Weight Diary" Windows™ software

### What You Need

- UC-321P Precision Personal Health Scale
- Connecting cable (included in the packaging)
- "Weight Diary" Windows™ software (included with the UC-321P)
- A personal computer with
  - Operating system: Windows 95 or later
  - Port: RS-232C port (9 pin)
  - Drive: CD-ROM drive

#### Note:

- Please refer to the instruction manual of your personal computer for the location of the RS-232C port.

WM:PD4000423A

### How to Connect the Cable

#### Caution:


- You should turn off your personal computer before connecting the cable.

There is a DATA PORT on the bottom of the UC-321P unit. ( as shown right)

1. Set the slide switch to "WEIGHT".
2. Insert the mini jack of the cable in the UC-321P.
3. Connect the other side of the cable (9 pin) to RS-232C port of your computer.
4. Then start your computer.

#### Notes:

- Keep the cable straight and keep the feet of the UC-321P off the cable.
- Gathering excess length of cable and putting under the scale is not recommended. This can cause the UC-321P to give an inaccurate reading .

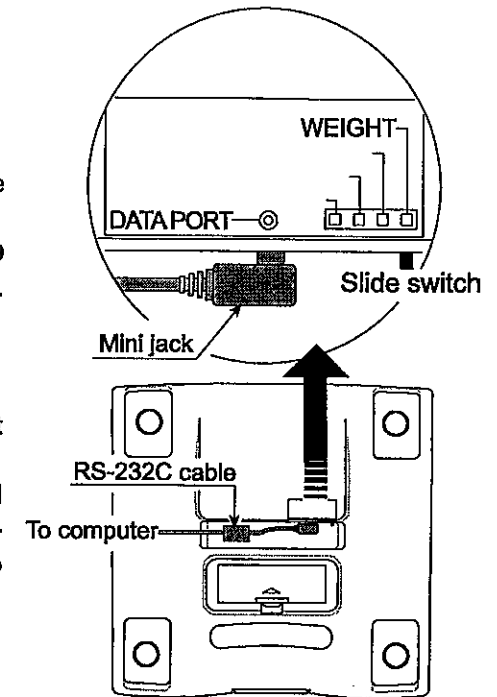

### Starting "Weight Diary" Software

- Run the Weight Diary from START/ PROGRAM/ Weight Diary/ Weight Diary.
- You will find the instruction manual in the CD-ROM and Help Menu of the Weight Diary.

#### A&D Co.,LTD.

1-243 Asahi, Kitamoto-shi, Saitama, 364-8585 JAPAN  
Telephone: [81] (485) 93-1111 Fax: [81] (485) 93-1119

#### A&D INSTRUMENTS LTD.

<Authorized Representative Established in the European Community>

Unit 24/26 Blacklands Way, Abingdon Business Park, Abingdon, Oxon OX14 1DY United Kingdom  
Telephone: [44] (1235) 550420 Fax: [44] (1235) 550485