UB-510

Wrist Blood Pressure Monitor

OPERATION MANUAL (English)

Manuel d'utilisation (French)

Manual de funcionamiento (Spanish)

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Manuale d'uso (Italian)

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Thank you for purchasing an A&D Monitor!

Operation Manual

Index

1. Getting Started	
- Set Components	2
- Device Description	2
- Installing the batteries	3
2. Measurement	
- Fastening the Cuff	4
- Measuring your Blood Pressure	4
- Memory Recall	5
- Tips for accurate Measurements	6


Getting Started

1. Set Components


This package contains the following items:

- Blood Pressure Monitor
- Instruction Manual
- Batteries (AAA) x 2
- Operation Manual
- Carrying Case

2. Device Description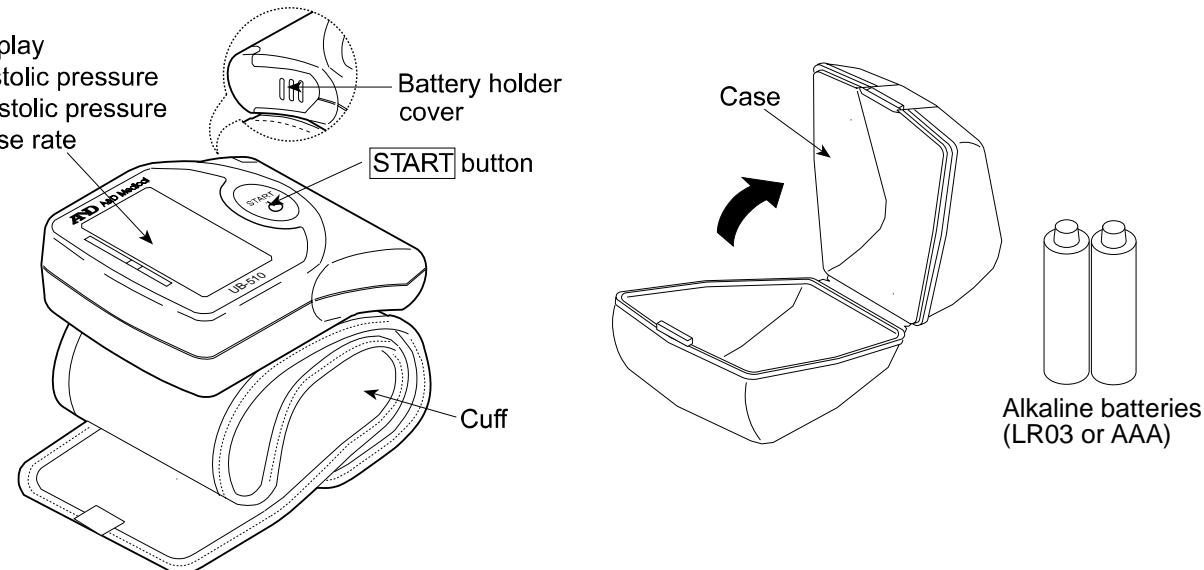


Display Description

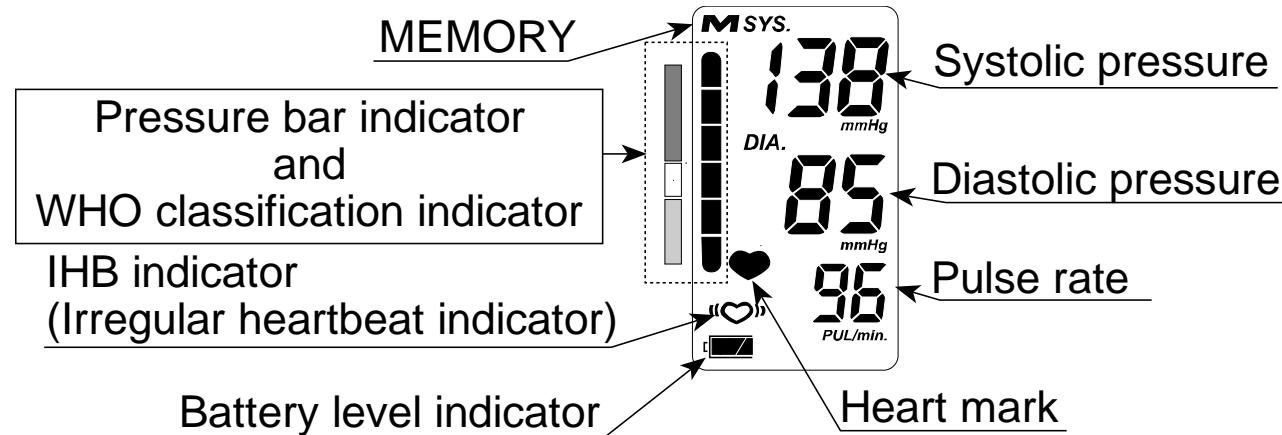3. Installing the Batteries

1. Slide the battery cover as shown to remove it from the device.
2. Insert two AAA / LR03 size batteries in the battery compartment, with the positive (+) and negative (-) terminals matching those indicated on the battery compartment.
3. Slide back the battery cover to close.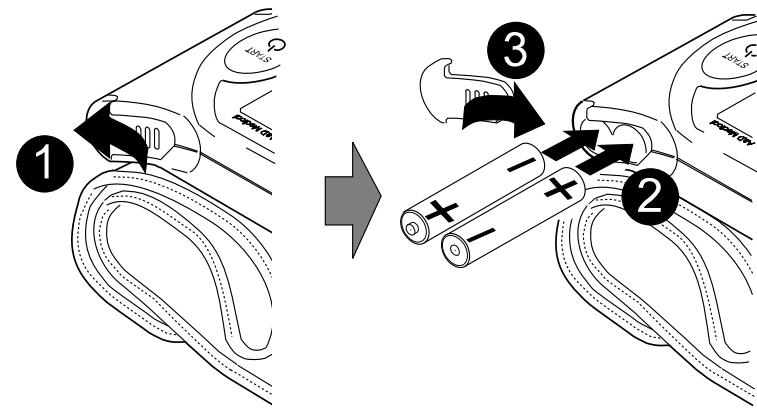


Measurement

1. Fastening the Cuff


1. Wrap the cuff around your wrist about 1 cm above your hand as shown in the figure on the right.
2. Fasten the cuff tightly using the velcro strip.

Note: For accurate measurements, fasten the cuff tightly and perform the measurement on a bare wrist.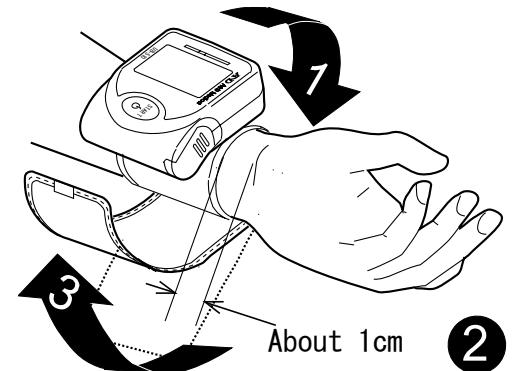

2. Measuring your blood pressure

1. Sit on a chair with your feet flat on the floor. Put your arm on a table and relax your posture.
The cuff should be about the same level as your heart.
2. Press the START button to start the measurement.
3. Keep quiet and still until the measurement is complete.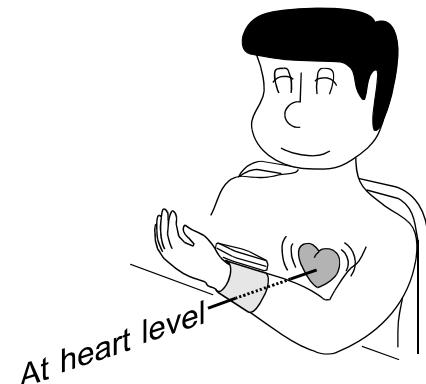


3. Memory Recall

This blood pressure monitor memorizes measurement data (Systolic, Diastolic and Pulse value) automatically after each measurement is complete.

- Total No. of memory: 30 sets
- When the memory reaches maximum capacity, it deletes the oldest memory data.
- All the memory data will be erased when the batteries are removed from the monitor.

1. Press the START button and keep the button depressed for approx. 3 seconds until the “n01” mark appears on the display.
2. Stop depressing the button.
3. All the data shall be displayed automatically. (Newest data will be displayed first)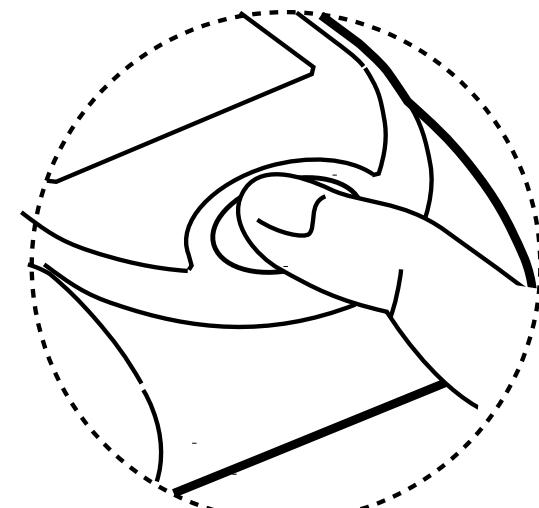

4. Tips for Accurate Measurements

Before measurement

- 1) Measure blood pressure at the same time each day; blood pressure fluctuates throughout the day.
- 1) Relax for five to ten minutes before taking your measurement.
- 2) Sit in a relaxed position.
- 3) Properly fasten the cuff on your arm as described above.


During the measurement

- 1) Do not talk.
- 2) Keep quiet and stay calm for the entire process.
- 3) Keep the cuff at heart level.

After the measurement

- 1) Do not become alarmed after each measurement result.
- 2) Consult your physician and avoid self-diagnosis.

Several factors that increase your blood pressure

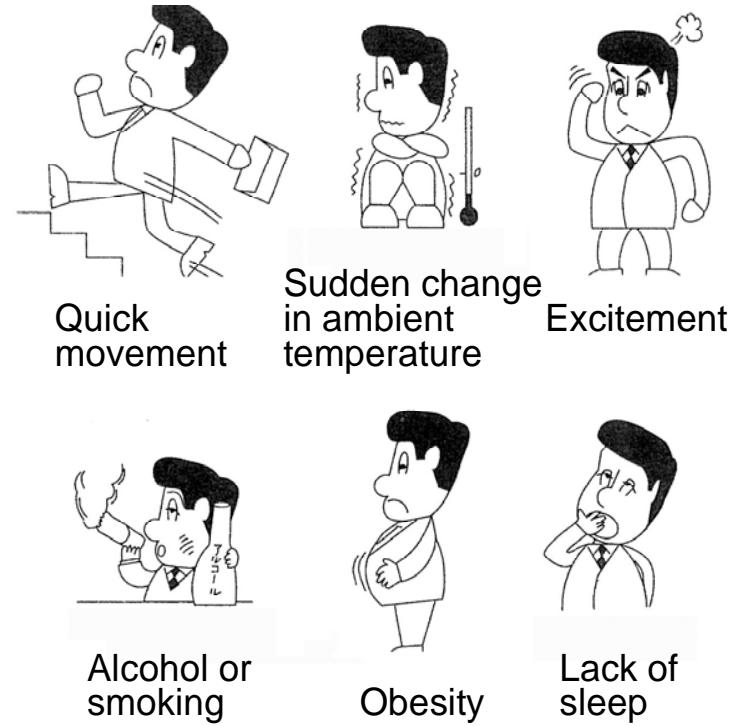For more details, please read the instruction manual included in the package.

Thank you! 谢谢您！ありがとうございます。 Grazie!
Danke! Grazie!
Obrigado! Merci! مرحباً Gracias! Grazie!
eyvallah!

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Copyright : A&D Company Limited, Tokyo, Japon
A&D Company, Limited