

Talking Blood Pressure Monitor



This monitor announces your blood pressure!

- · Easy one-button operation.
- Systolic, diastolic and heart rate are announced when measurement is complete.
- · Easy-to-read display shows blood pressure and heart rate simultaneously.
- Oscillometric method ensures hospital accuracy for everyday use.





Talking Blood Pressure Monitor





Easy one-button operation.

A single large button activates the following functions: power on, start, cuff inflation, blood pressure measurement, announcement, cuff deflation, and power off.

Systolic, diastolic and heart rate are announced when measurement is complete.

Measurement results are clearly and pleasantly announced. The volume may be adjusted using the slide on the lower left front of the monitor. The unit shuts off automatically after announcements of the measurement or may be stopped at any time by pressing the start button. The UA-767 Voice includes a jack for headphones (not included).

Easy-to-read, large character display.

After each measurement, the UA-767 Voice simultaneously displays systolic and diastolic pressures and heart rate on large, clear 16mm character display.

Oscillometric method ensures hospital accuracy for everyday use.


The UA-767 Voice utilizes the same Oscillometric technique used by hospital bedside monitors and the UA-767 Auto-inflation Blood Pressure Monitor, assuring reliable and accurate blood pressure measurement at home.

Specifications

Checken and	
(BPM Section)	
Method	Oscillometric
Display type	Digital, LCD
Measurement Range	20~280mmHg(Pressure) 40~200 pulse/minute (pulse)
Accuracy	3mmHg or 2%, whichever greater(Pressure) 5% (Pulse)
Pressurization	Automatic pressurization by micro pump
Depressurization	Automatic constant speed exhaust valve
Power supply	4 type AA batteries (SUM-3/R6P)
Battery life	Approximately 200 measurements
Operating environment	+10C ~ + 40C (+50F ~ + 104F), less than 85% RH
Storage environment	-20C ~ + 70C (- 4F ~ + 158F), less than 95% RH
Weight	Approximately 340g (0.8lbs.)
Dimensions	66.7(H)x163.7(W)x111(D)mm 2.6(H)x6.4(W)x4.4(D)inches
(Voice Section)	
Language	English
Number	1~20/30/40/50/60/70/80/90/100/200
words	your/is/blood pressure/over/pulse/and/error/re-measure/ per minute/beets

World Health Organization Blood Pressure Standards

At present the WHO(World Health Organization) has set up four classifications of blood pressure which are not related to the subject's age. They are outlined in the chart.

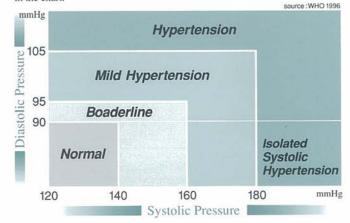



3-23-14 Higashi-Ikebukuro,Toshima-ku,Tokyo 170-0013 Japan Telephone:[81](03)5391-6132 Fax:[81](03)5391-6148

1-243, Asahi,Kitamoto-shi,Saitama, 364-8585 Japan Telephone:[81](0485)93-1111 Fax:[81](0485)93-1119 (EN46001 Certified Factory)

http://www.aandd.co.jp

A&D Medical

1555 McCandless Drive, Milpitas, CA. 95035 U.S.A. Telephone:[1](408)263-5333 Fax:[1](408)263-0119

A&D INSTRUMENTS LTD.

A&D MERCURY PTY, LTD.

Abingdon Science Park, Abingdon, Oxford OX14 3YS ENGLAND Telephone: [44](1235)550-420 Fax: [44](1235)550-485 (Authorized Representative Established in the European Community) 32 Dew Street, Thebarton, South Australia 5031 AUSTRALIA Telephone: [61](088)352-3033 Fax: [61](088)352-7409

REF * ADC-UA767V-000603-02PR1